**Indicatie**

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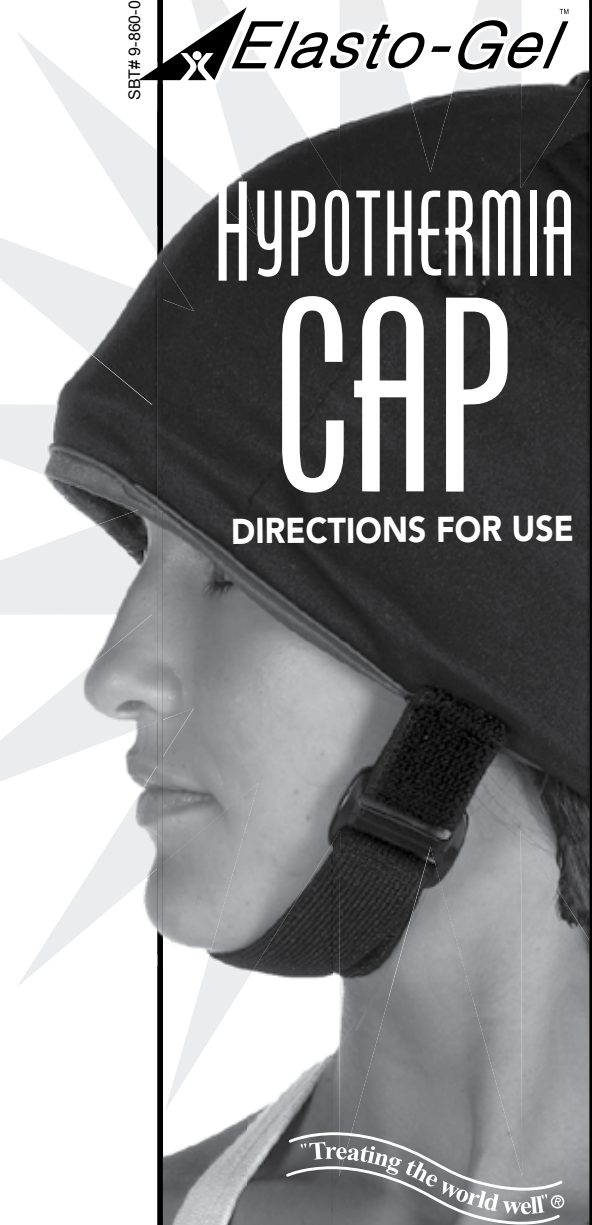
© Rev. E 6-08  
 SBT# 9-860-012

For additional replacement liners contact Southwest Technologies at email: info@elastogel.com or phone: 800-247-9951; (816) 221-2442 fax: (816) 221-3995(USA)

**HYPOTHERMIA  
 CAP**  
 DIRECTIONS FOR USE

"Treating the world well"®

Made in the USA by  
**southwest technologies inc.**  
 1746 Levee Road, North Kansas City, MO 64116  
 phone: (800) 247-9951 phone: (816) 221-2442 fax: (816) 221-3995  
 email: info@elastogel.com • website: www.elastogel.com

## Elasto-Gel™ HYPOTHERMIA CAP (English)

### Indication

The Elasto-Gel™ Hypothermia Cap reduces alopecia as a result of Chemotherapy treatments such as Docetaxel(Taxotere®).

### Contra-Indication

- Cold intolerance

### Product Description

The Elasto-Gel™ hypothermia cap is made of a glycerine-based gel that maintains its suppleness (or flexibility) even at temperatures of -20°C to -30°C. The cover material is a water resistant stretch-able fabric. The chin strap assures good position maintenance.

### Directions For Use

- Keep the cap in a plastic bag in a freezer at -18° to -21°C (0° to -5°F) for a minimum of 4 hours (12 hours preferred). The cap may be left in the freezer indefinitely until ready for use. The cap will remain flexible even at -30°C(-20°F).
- Moisten hair with cold water.
- Use the inner disposable liner for hygienic purposes and to avoid direct contact with the cold.
- Apply the cap 15 minutes before the start of the treatment and leave it on until 15 minutes after the end of the treatment. For an infusion duration of about 1 hour to 1.5 hour, 2 caps may be necessary.

### Important Notes

For best results we recommend changing the cap every 30 to 45 minutes. Cap changes must be accomplished as quickly as possible to avoid a vasodilation reflex. The cap must be worn for the entire therapy session as an interruption could compromise their effectiveness. Protect exposed skin, the ears in particular when using cold therapy. After use, allow the cap to dry before replacing it in the freezer.

### Cleaning Instructions

***DO NOT GET THE PRODUCT WET.*** The gel absorbs moisture.

### Guarantee

Elasto-Gel™ hypothermia cap is guaranteed for 6 months from date of purchase against manufacturing defects.

<div><span><span></span></span></div>	<div><span><span></span></span></div>	<div><span><span></span></span></div>	<div><span><span></span></span></div>	<div><span><span></span></span></div>	<div><span><span></span></span></div>
<div><span><span></span></span></div>	<div><span><span></span></span></div>	<div><span><span></span></span></div>	<div><span><span></span></span></div>	<div><span><span></span></span></div>	<div><span><span></span></span></div>

<b>Distributed by</b>	<b>Date of Purchase</b>
<div><span><span></span></span></div>	<div><span><span></span></span></div>
<div><span><span></span></span></div>	<div><span><span></span></span></div>
<div><span><span></span></span></div>	<div><span><span></span></span></div>
<div><span><span></span></span></div>	<div><span><span></span></span></div>

M. Devices Group - Marlborough House - Riding Street - Southport - PR8 1EW - United Kingdom - Tel: +44 1704 544 944 - Fax: +44 1704 544 050

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