  
**HYPOTHERMISCHE WANTEN**  
(Dutch)

**Indicatie**

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SBT# 9-860-008 © Rev. C 10-06

For additional replacement liners contact Southwest Technologies at email: info@elastogel.com or phone: 800-247-9951; (816) 221-2442 fax:(816) 221-3995(USA)

**HYPOTHERMIA  
GLOVES**

**DIRECTIONS FOR USE**

"Treating the world well"®

**southwest technologies inc.**  
1746 Levee Road, North Kansas City, MO 64116  
phone: (800) 247-9951 phone: (816) 221-2442 fax: (816) 221-3995  
email: info@elastogel.com • website: www.elastogel.com

## HYPOTHERMIA GLOVES

(English)

### Indication

Prevention of onycholysis and skin toxicity resulting from treatments with Docetaxel(Taxotere®).

### Contra-Indication

- Raynaud's disease
- Distal metastasis
- Distal arteriopathy
- Cold intolerance

### Product Description

The Elasto-Gel™ hypothermia gloves are made of a glycerine-based gel that maintains its suppleness (or flexibility) even at temperatures of -20°C to -30°C. The cover material is a water resistant stretch-able fabric. The wrist strap assures good position maintenance.

### Directions For Use (\*)

1. Keep the gloves in a plastic bag in a freezer at -18° to -21°C (0° to -5°F) for a minimum of 4 hours (12 hours preferred). They may be left in the freezer indefinitely until ready for use. They will remain flexible even at -30°C(-20°F).
2. Use the nonwoven disposable liner for hygiene purpose and to avoid direct contact with the cold.
3. Apply the gloves 15 minutes before the start of the treatment and leave them on until 15 minutes after the end of the treatment. For an infusion duration of about 1 hour to 1.5 hour, 2 pairs of gloves may be necessary.

(\*) Dr. F. Scotté, et al., Journal of Clinical Oncology, Vol.23, N° 19 (1 July 2005), 4424-4429 "Multicenter Study of a Frozen Glove to Prevent Docetaxel-Induced Onycholysis and Cutaneous Toxicity of the Hand"

### Important Notes

For best results we recommend changing the gloves every 30 to 45 minutes. Glove changes must be accomplished as quickly as possible to avoid a vasodilation reflex. The gloves must be worn for the entire therapy session as an interruption could compromise their effectiveness. After use, allow the gloves to dry before replacing them in the freezer.

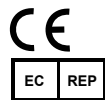
### Cleaning Instructions

Remove outside cover and wash with mild soap and water. Air dry the cover. DO NOT GET THE GEL INSERT WET.

### Guarantee

Elasto-Gel™ hypothermia gloves are guaranteed for 6 months from date of purchase against manufacturing defects.

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RaMa Medical AB