HYPOTHERMISCHE SOKKEN
(Dutch)

Indicatie

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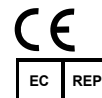
DIRECTIONS FOR USE

"Treating the world well"®

 **southwest technologies inc.**
1746 Levee Road, North Kansas City, MO 64116
phone: (800) 247-9951 phone: (816) 221-2442 fax: (816) 221-3995
email: info@elastogel.com • website: www.elastogel.com

HYPOTHERMIA SLIPPERS

(English)

Indication

Prevention of onycholysis and skin toxicity resulting from treatments with Docetaxel(Taxotere®).

Contra-Indication

- Raynaud's disease
- Distal metastasis
- Distal arteriopathy
- Cold intolerance

Product Description

The Elasto-Gel™ hypothermia slippers are made of a glycerine-based gel that maintains its suppleness (or flexibility) even at temperatures of -20°C to -30°C. The cover material is a water resistant stretchable fabric. The ankle strap assures good position maintenance.

Directions For Use (*)

1. Keep the slippers in a plastic bag in a freezer at -18° to -21°C (0° to -5°F) for a minimum of 4 hours (12 hours preferred). They may be left in the freezer indefinitely until ready for use. They will remain flexible even at -30°C(-20°F).
2. Use the "disposable under-slippers" or "socks" for hygiene purpose and to avoid direct contact with the cold.
3. Apply the slippers 15 minutes before the start of the treatment and leave them on until 15 minutes after the end of the treatment. For an infusion duration of about 1 hour to 1.5 hour, 2 pairs of slippers may be necessary.

(*) Dr. F. Scotté, et al., Journal of Clinical Oncology, Vol.23, N° 19 (1 July 2005), 4424-4429 "Multicenter Study of a Frozen Glove to Prevent Docetaxel-Induced Onycholysis and Cutaneous Toxicity of the Hand"
"A specific study for the Feet is currently running."

Important Notes

For best results we recommend changing the slippers every 30 to 45 minutes. Slipper changes must be accomplished as quickly as possible to avoid a vasodilation reflex. The slippers must be worn for the entire therapy session as an interruption could compromise their effectiveness.

After use, allow the slippers to dry before replacing them in the freezer.

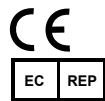
Cleaning Instructions

Wipe the cover down with a damp(not wet) cloth using mild soap or disinfectant. Allow the product to fully air dry before continued use. DO NOT GET THE GEL INSERT WET.

Guarantee

Elasto-Gel™ hypothermia slippers are guaranteed for 6 months from date of purchase against manufacturing defects.

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© Rev. C. 10-06
RaMa Medical AB