 southwest technologies inc.

MANUFACTURER

1746 Levee Road, North Kansas City, MO 64116  
ph:(800) 247-9951 ph:(816) 221-2442 fax:(816) 221-3995  
email: info@elastogel.com • website: www.elastogel.com

Distributed By:

CONSULT  
INSTRUCTIONS

KEEP DRY

DO NOT REUSE

DO NOT USE  
IF DAMAGED

© Rev. A 5-13

# DIRECTIONS FOR USE

STERILE | R

 **Nectacare**<sup>®</sup>

*Contains Authentic Medical Grade Manuka Honey  
from New Zealand*

**MANUKA HONEY  
WOUND DRESSING**

*"Treating the world well"<sup>®</sup>*

**DR4300** 2" x 3" Manuka Honey Dressing

**DR4200** 4" x 4" Manuka Honey Dressing

(English)

## Nectacare® Honey Wound Dressing Directions For Use

### **DRESSINGS ARE STERILE UNTIL OPENED**

1. Select a dressing size that will allow the dressing to extend 1" to 2" beyond the wound or is at least 25% larger than the wound.
2. Prepare the wound site by cleansing as needed.
3. Remove the sterile dressing from the package.
4. If cutting is necessary, leave the clear plastic cover sheet in place.
5. To apply the dressing, remove the clear plastic cover sheet from the gel. DO NOT remove the white stretch backing material.
6. Cover the wound by placing the gel side of the dressing directly on the wound.
7. Dressing must be secured well. The dressing may be secured by stretch bandages, elastic wrap or tape.
8. **CHECKING THE WOUND:** This can easily be done by gently lifting one corner of the dressing then pulling the whole dressing back until the wound is visible. As Nectacare® Manuka Honey Dressing becomes saturated, it will look somewhat slippery and rubbery around the edges. This is a sign that the dressing needs to be changed.
9. Remove before showers, baths and swimming.

### **Contraindications**

- Allergy or sensitivity to honey or bee products.
- Infection could occur. **Do not re-use.**

**Caution: Consult your physician if signs of infection occur: redness, swelling, fever, etc.**

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