

Stimulen™ enhanced collagen woundcare



collagen gel

A gel formulated to produce a highly concentrated

dispersion of modified collagen. This gel provides a method to apply these collagens to wounds when it is difficult to apply the powder, such as a leg in a vertical position or the bottom of the foot. It can also be used to fill a cavity wound.

This gel contains a high level of glycerine. Cover the gel and wound area with an appropriate secondary dressing. We recommend that you use Elasto-Gel™ gel sheets. Elasto-Gel™ gel sheets containing high levels of glycerine have been shown to exhibit bacteriostatic properties.



Instructions for Use

1. **Tubes:** Before use remove cap and peel off the safety seal.
Packets: Tear corner off of the packet where packet is indented.
2. Cleanse the wound using standard protocol.
3. Apply a generous coating of the collagen gel onto wound site, or fill wound cavity.
4. Cover the wound and Stimulen™ with a non-adherent secondary dressing*, and secure with tape or appropriate covering.
5. Change the secondary dressing* and re-apply Stimulen™ collagen gel daily or as needed.

Ordering Information

Stock #	Description
ST9502	0.5 oz collagen gel, 12 tubes/cs
ST9503	1 oz collagen gel, 12 tubes/cs
ST9504	3 oz collagen gel, 6 tubes/cs
ST9506	5 gr collagen gel, 10 packets/box, 10 boxes/cs

Ask about our other Stimulen™ products

Stimulen™

enhanced collagen woundcare



- Specially Prepared Polypeptides and Modified Collagen
- Forms a Protective Gel on Wound Bed
- Gel Protects New Cells and Tissue from Dehydration
- Provides a Moist Environment to Promote Rapid Healing



southwest technologies inc.

1746 Levee Road, North Kansas City, MO. 64116
phone:(800) 247-9951 phone:(816) 221-2442 fax:(816) 221-3995
email: info@elastogel.com • website: www.elastogel.com