

# Stimulen™

enhanced collagen moisturizing gel



## ***lotion***

This product is a thixotropic liquid that will form a gel when kept undisturbed for several hours at temperatures below 72° F(21°C). It fluidizes (becomes liquid) immediately with agitation or at slightly higher temperatures or when applied to the skin. It is designed to moisturize and condition the skin. It may improve skin problems such as: skin rash or irritation caused by various diseases or exposure to various chemicals, soaps and detergents, microbes, or other common problems -- dry hands, feet, elbows, athletes foot, jock itch, etc. It may be used to soften the skin that may be causing wrinkles, callouses, and corns. It may be effective for improving scar remodeling.



## **Instructions for Use**

1. **Tubes:** Before use remove cap and peel off the safety seal.  
**Packets:** Tear corner off of the packet where packet is indented.
2. Cleanse the target area using standard protocol.
3. Apply 2 to 4 drops of the lotion and spread evenly over the affected area.

## **Ordering Information**

Stock #	Description
ST9555	2 oz lotion, 12 btls/cs
ST9552	5 gr lotion, 10 packets/box, 10 boxes/cs

Ask about our other Stimulen™ products

# Stimulen™

enhanced collagen woundcare



- Specially Prepared Polypeptides and Modified Collagen
- Forms a Protective Gel on Wound Bed
- Gel Protects New Cells and Tissue from Dehydration
- Provides a Moist Environment to Promote Rapid Healing



**southwest technologies inc.**

1746 Levee Road, North Kansas City, MO. 64116  
phone:(800) 247-9951 phone:(816) 221-2442 fax:(816) 221-3995  
email: info@elastogel.com • website: www.elastogel.com