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(German)

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Directions for Use

(English)

ⓘ When Applying After Toe-Nail Removal or to an Open Wound:

1. Prepare wound site by cleaning as needed.
2. Remove dressing from the package.
3. Apply dressing as shown in diagram.
4. Change Dressing as Needed: If leaking occurs and/or if the dressing becomes highly saturated with exudate.
5. Do Not get the gel wet.
6. Consult a physician if signs of infection occur; redness, swelling, fever, etc.

Ⓜ When Applying to Intact Skin or Directly to Toe-Nail:

The dressing may be left in place for seven (7) days – be sure to make a waterproof seal to protect the gel from water. The product may be worn in the shower if the tape is properly secured. Toe-Aid™ is a unique water-resistant adhesive system with a glycerine pad attached. Toe-Aid™ helps control factors that can inhibit the healing process.

DIRECTIONS FOR USE

DR8450

1.25" x 1.25" • 3cm x 3 cm
Gel Pad Size

Made in the USA

"Treating the world well"®

STERILE R

Ask about other advanced
Hot/Cold Therapy Products
and Wound Care Products
available from
Southwest Technologies

Manufactured by

 southwest
technologies inc.

1746 Levee Road, North Kansas City, MO 64116
ph: (800) 247-9951 ph: (816) 221-2442 fax: (816) 221-3995
email: info@elastogel.com • website: www.elastogel.com

Distributed by

0086 M. Devices Group
Marlborough House Riding Street Southport PR8 1EW U.K.
Tel: +44 1704 544 944 Fax: +44 1704 544 050

© SWT# 9-860-004 Rev. C 8-10

Technologies Beyond Definition • Applications Beyond Imagination

From the manufacturers of
Elasto-Gel

T-SHAPED GEL BANDAGES™
ToeAid™
FOR DIGITS and MORE

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