

HC-805 8"x16" HOT/COLD PACK

***CAUTION: DO NOT OVERHEAT!!** Overheating the product can cause the plastic cover to melt. Due to the nature of the gel, it may only feel warm to the touch, but if the pack has been overheated, too much heat may be transferred to the body and can cause burns.

The ideal hot/cold therapy.

The **Elasto-Gel™** Pack can be used for either hot or cold therapy. The pack will provide a dry heat therapy, due to the plastic cover on the gel; however, wet towels may be used underneath the packs to provide a moist heat. The product will hold the heat or cold for 20-40 minutes.

A specially formulated gel:

This product is made with **Elasto-Gel™**, a patented* gel with exceptional properties for transferring heat or soothing cold. If puncture occurs, the pack will not leak but use with wet towels should be discontinued.

DIRECTIONS FOR USE

1. COLD THERAPY:

Store the pack in the freezer for a minimum of two (2) hours. It may be left in the freezer indefinitely until ready to use. It will remain flexible and does not get hard even at -20°F (-30° C). **Note: Always store product in a plastic bag while in freezer.**

2. HEAT THERAPY:

The most convenient method of heating the pack is in a **microwave oven**. Suggested heating times: **600-700 watt microwave:** 45-60 Seconds per side, **1100 watt:** 45 Seconds per side. **Note: all microwaves vary in heating and may have "hot spots". Be cautious when heating.**

Wait one minute (60 seconds) before use. The pack should feel **WARM** to the touch, **NOT HOT**. Hold the pack firmly against the skin for 30 seconds **before re-heating***. If the pack is not warm enough, heat in 20 second increments and rotate the product each time.

The packs can also be heated in a **conventional oven** at 250°F for 10-15 minutes. Place pack on cookie sheet or baking dish. The outside cover can be removed and the pack can also be heated in a **water bath** for 10-15 minutes.

3. CLEANING INSTRUCTIONS

Remove the outside cover and hand wash in cold water and mild soap. Hang up to dry. Wipe the inside urethane bladder with a damp cloth using mild soap or disinfectant.



Treating the World Well®
southwest technologies, inc.
1746 Levee Road • North Kansas City • MO 6 4116
Phone: (816) 221-2442 • Toll Free: (800) 247-9951
Fax: (816) 221-3995 • e-mail: info@elastogel.com
Visit our Website: www.elastogel.com



MANUFACTURER



EU Authorized Representative:

M. Devices Group
Marlborough House • Riding Street
Southport • PR8 1EW • UK
Phone: 00 44 (0) 1704 544 944
Fax: 00 44 (0) 1704 544 050
e-mail: mdg@tour.demon.co.uk