

# TW-6002 6"x16" Therapy Wrap

**\*CAUTION: DO NOT OVERHEAT!!** Due to the nature of the gel, it may only feel warm to the touch, but if the wrap has been overheated, too much heat may be transferred to the body and can cause burns.

## **The ideal hot/cold therapy.**

The **Elasto-Gel™** Therapy Products are designed to provide **moist heat** therapy or **soothing cold** therapy for relief of minor muscular and joint aches and pains. The hot or cold therapy will last 20 to 40 minutes per application. The product is held in place with hook and loop type closures, which allow the user to move about during therapy. The product is microwavable so it can provide nearly continuous heat therapy. When used cold, the product conforms to contours, even at -20°F (-30°C), allowing comfortable soothing cold applications.

## **A specially formulated gel:**

This product is made with **Elasto-Gel™**, a patented\* gel with exceptional properties for transferring heat or soothing cold. If punctured, it will not leak and will remain usable. The unique combination of specially selected fabric and tough flexible gel provides an exceptional durable product for use in hospitals, rehabilitation centers, athletic facilities and the home.

## **DIRECTIONS FOR USE**

### 1. **HEAT THERAPY:**

The most convenient method of heating the wrap is in a **microwave oven**. Suggested heating times: **600-700 watt microwave:** 30-45 Seconds per side, **1100 watt:** 35 Seconds per side.

**Note: all microwaves vary in heating and may have "hot spots". Be cautious when heating.**

Wait one minute (60 seconds) before using. The wrap should feel **WARM** to the touch, **NOT HOT**. Hold the wrap firmly against the skin for 30 seconds before re-heating. If the wrap is not warm enough, heat in **15 second increments** and rotate the product each time.

The wrap can also be heated in a **conventional oven** at 250°F for 10-15 minutes. Place wrap on a cookie sheet or baking dish in the center of the oven. Do not allow straps to touch the heating element. Flip the wrap after 6 minutes for even heating.

### 2. **COLD THERAPY:**

Store the wrap in the freezer for a minimum of two (2) hours. It may be left in the freezer indefinitely until ready to use. It will remain flexible even at -20°F. Remove the wrap prior to defrosting the freezer. **Note: Always store products in a plastic bag while in freezer.**

### 3. **DO NOT GET THE PRODUCTS WET:**

The gel will absorb water and swell. If the product becomes damp, lay it on a flat surface and allow it to fully air dry before continuing use. Do not use wet towels under the gel.

### 4. **CLEANING INSTRUCTIONS:**

Remove the outside cover and hand wash in cold water and mild soap. Hang up to dry. Wipe the white inner cover with a damp (not wet) cloth using mild soap or disinfectant. Allow the product to fully air dry before continuing to use.



Treating the World Well®  
**southwest technologies, inc.**

1746 Levee Road • North Kansas City • MO 6 4116

Phone: (816) 221-2442 • Toll Free: (800) 247-9951

Fax: (816) 221-3995 • e-mail: info@elastogel.com

Visit our Website: www.elastogel.com

MANUFACTURER



**EU Authorized Representative:**

**M. Devices Group**

Marlborough House • Riding Street

Southport • PR8 1EW • UK

Phone: 00 44 (0) 1704 544 944

Fax: 00 44 (0) 1704 544 050

e-mail: mdg@tour.demon.co.uk